

CYTOTOXIC SPILL KIT INSTRUCTIONS

CONTENTS

PPE

- Hair net
- 2 pairs gloves
- Overshoes
- Gown
- P2 mask & safety goggles

ABSORBENTS

- Blue Pad
- Absorbent Powder
- MAX60 wipes

CLEAN UP MATERIALS

- Scoop and scraper
- Cytotoxic waste bag
- Clinical Detergent
- "Caution Spill" signs
- WFI & cable tie
- Water 100ml
- Cable ties



FIRST STEPS WHEN A SPILL OCCURS

1. Inform nearby personnel that a spill has occurred.
2. Retrieve the nearest Cytotoxic Spill Kit.
3. Remove the seal and position the "Caution Spill" signs.
4. Put on personal protection equipment contained in the spill kit.
5. Make sure that the gloves are over the cuff of the gown.
6. Once you have prevented people from entering the area and correct PPE is worn, you are ready to address the spill.



CYTOTOXIC SPILL CLEAN UP

1. Remove all of the items from the spill kit and place them in an area unaffected by the spill.
2. Open the cytotoxic waste bag and roll the sides down until it is self supporting and place on the ground.
3. Place the open spill kit pail inside the waste bag.
4. Carefully collect any pieces of glass using the scoop + scraper and dispose in the pail.
5. For powder spill – place wipe over powder. Gently pour water over wipe to dissolve powder and dispose.
6. For liquid spill – gently place the blue pad, plastic side up, on the spill. Allow 2 minutes for absorption of liquids.
7. Dispose of blue pad in the pail.
8. Apply absorbent powder to any remaining liquid or semi-solid spill.
9. Allow 2-3 minutes for the liquid to be converted into an easily managed gel.
10. Collect with the scoop and scraper and dispose of in the pail.
11. Apply some clinical detergent to the area of the spill, wipe dry with the wipes and dispose of in the pail.
12. Once the area is clean and dry, remove overshoes and outer glove and dispose of in the pail.
13. Replace the lid of the pail.
14. Retrieve the “Caution Spill” signs and place them on the pail lid.
15. Remove hairnet, gown, mask, overshoes and then inner gloves and place on the pail lid.
16. Carefully roll up the bag touching only the outside and seal with the cable tie.
17. Dispose of in accordance with local, state and federal regulations.
18. Once the contaminated waste is placed in the designated disposal location, inform your supervisor that the spill kits has been used to ensure that it is replaced.



TO RE-ORDER SPILL KITS

Fax: **03 9565 4488**

Email: **info@sladehealth.com.au**

www.sladehealth.com.au



The expertise hospitals rely on